

<b>REF</b> 513-2	6x3mL	<b>EXP</b> 2024-03-22	<b>LOT</b> 30F1C30	更新日期： 2020-11 1 / 2
------------------	-------	-----------------------	--------------------	------------------------

批号：30F1C30				
项目\仪器\试剂方法	单位	均值	+ / - 2 SD	
<b>(AFP)甲胎蛋白</b>				
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	94.3	75.4-113	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	94.7	75.8-114	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	78.7	63-94.4	
<b>(CA 125)糖类抗原 125</b>				
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	284	227-341	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	334	267-401	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	182	146-218	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	217	174-260	
<b>(CA 15-3)糖类抗原 15-3</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	75.8	60.6-91	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	114	91.2-137	
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	97.8	78.2-117	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	125	100-150	
<b>(CA 19-9)糖类抗原 19-9</b>				
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	204	163-245	
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	237	190-284	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	482	386-578	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	100	80-120	
<b>(CA 50)糖类抗原 50</b>				
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA )	IU/mL	68.1	51.7-84.5	
<b>(CA 72-4)糖类抗原 72-4</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	13.4	10.7-16.1	
<b>(CA242)糖类抗原 242</b>				
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA )	U/mL	56.6	43-70.2	
<b>(CEA)癌胚抗原</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	51.1	40.9-61.3	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	53.9	43.1-64.7	
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	46.7	37.4-56	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	68.1	54.5-81.7	
<b>(Cyfra 21-1)细胞角蛋白19片段</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	12.2	9.76-14.6	
<b>(Ferritin)铁蛋白</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	µg/L	304	243-365	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	µg/L	488	390-586	
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	µg/L	293	234-352	

<b>REF</b> 513-2	6x3mL	<b>EXP</b> 2024-03-22	<b>LOT</b> 30F1C30	更新日期： 2020-11 2 / 2
------------------	-------	-----------------------	--------------------	------------------------

批号：30F1C30				
项目\仪器\试剂方法	单位	均值	+ / - 2 SD	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	µg/L	307	246-368	
<b>(F-PSA)游离前列腺特异性抗原</b>				
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	7.11	5.69-8.53	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	4.64	3.7-5.58	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	4.88	3.9-5.86	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	5.12	4.1-6.14	
<b>(hCG)人绒毛膜促性腺激素</b>				
Siemens ADVIA Centaur XP Dedicated \ Enzyme linked immunosorbent assays (ELISA)	IU/L	110	88-132	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	153	122-184	
<b>(HE4)人附睾蛋白4</b>				
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	383	306-460	
<b>(PSA)前列腺特异性抗原</b>				
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	16.1	12.9-19.3	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	18.9	15.1-22.7	
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	19.7	15.8-23.6	
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	20.1	16.1-24.1	
<b>(SCC)鳞状细胞癌抗原</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	15.5	12.4-18.6	
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA )	ng/mL	14.3	10.9-17.7	
<b>(TPA)组织多肽抗原</b>				
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	U/L	8030	6420-9630	
<b>(TPS)组织多肽特异抗原</b>				
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence Immunoassay (CLIA )	U/L	262	210-314	

注：此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来，仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。