

<b>REF</b>	521-1	6x3mL	<b>EXP</b>	2025-01-01	<b>LOT</b>	31B2B49	更新日期:	2024-08
								1 / 3

		批号: 31B2B49			
项目\仪器\试剂方法	单位	均值 \ CV		Lab \ 点数	
<b>(Alb)白蛋白</b>					
Beckman AU series	g/L	19.7	1%	<5	<100
Beckman Coulter \ Bromocresol green					
Roche cobas 4000/6000/8000	g/L	20.3	12%	<5	<100
Roche \ Bromocresol green					
<b>(ASO)抗链球菌溶血素 O 抗体</b>					
Mindray BS series	IU/mL	152	3.4%	8	382
MedicalSystem \ OTHER					
Siemens ADVIA 1800/2400	IU/mL	138	5%	29	3166
DiaSys \ Immunoturbidimetry (ITA)					
Beckman Immage 800	IU/mL	94.7	17.5%	11	536
Beckman Coulter \ Immunonephelometry (INA)					
Roche Hitachi/Hitachi Systems	IU/mL	205	8%	10	522
Whitman \ Immunoturbidimetry (ITA)					
Beckman AU series	IU/mL	116	9.9%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
Siemens BN ProSpec	IU/mL	144	1.8%	<5	<100
Siemens \ Immunonephelometry (INA)					
Mindray BS series	IU/mL	105	1.5%	<5	<100
Mindray \ Immunoturbidimetry (ITA)					
KHB Polaris i2400	IU/mL	167	9.9%	<5	<100
KHB \ Chemiluminescence Immunoassay (CLIA)					
<b>(AT-III)抗凝血酶III</b>					
Stago STA series	%	7	12%	<5	<100
Dedicated \ Nephelometry					
<b>(C3)补体 C3</b>					
Mindray BS series	mg/dL	67.2	4%	8	386
MedicalSystem \ OTHER					
Siemens ADVIA 1800/2400	mg/dL	77	2.6%	38	4298
DiaSys \ Immunoturbidimetry (ITA)					
Beckman Immage 800	mg/dL	71.6	8.9%	<5	190
Beckman Coulter \ Immunonephelometry (INA)					
KHB Polaris i2400	mg/dL	82	9.9%	<5	<100
KHB \ Chemiluminescence Immunoassay (CLIA)					
Beckman AU series	mg/dL	77	9.9%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
<b>(C4)补体 C4</b>					
Mindray BS series	mg/dL	18.6	3.4%	8	388
MedicalSystem \ OTHER					
Siemens ADVIA 1800/2400	mg/dL	16.5	3.2%	38	4292
DiaSys \ Immunoturbidimetry (ITA)					
Beckman Immage 800	mg/dL	18.5	8.1%	<5	148
Beckman Coulter \ Immunonephelometry (INA)					
Beckman AU series	mg/dL	19	0.4%	<5	<100
chongqiong zybion \ Immunoturbidimetry (ITA)					
KHB Polaris i2400	mg/dL	19	9.9%	<5	<100
KHB \ Chemiluminescence Immunoassay (CLIA)					
Beckman AU series	mg/dL	20	9.9%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
<b>(Cer)铜蓝蛋白</b>					
Beckman Immage 800	g/L	0.132	12%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
<b>(CRP)C反应蛋白</b>					
Mindray BS series	mg/dL	1.5	7.5%	8	436
VISIONBIO \ Immunoturbidimetry (ITA)					
Mindray BS series	mg/dL	1.43	3.9%	8	192
MedicalSystem \ OTHER					
Beckman Immage 800	mg/dL	1.29	10.4%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
TOSHIBA TBA	mg/dL	1.23	4.8%	<5	<100
TOSHIBA \ Immunoturbidimetry (ITA)					
Siemens BN ProSpec	mg/dL	1.54	3.7%	<5	<100
Siemens \ Immunonephelometry (INA)					
<b>(Ferritin)铁蛋白</b>					
Roche cobas 4000/6000/8000	ng/mL	66.4	12%	<5	<100
Roche \ Immunoturbidimetry (ITA)					
<b>(IgA)免疫球蛋白 IgA</b>					
Siemens ADVIA 1800/2400	mg/dL	130	3.3%	38	4284
DiaSys \ Immunoturbidimetry (ITA)					
Beckman AU series	mg/dL	119	1.8%	<5	<100
chongqiong zybion \ OTHER					
Beckman Immage 800	mg/dL	119	6.9%	<5	140
Beckman Coulter \ Immunonephelometry (INA)					
KHB Polaris i2400	mg/dL	144	9.9%	<5	<100
KHB \ Chemiluminescence Immunoassay (CLIA)					
Beckman AU series	mg/dL	135	9.9%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
<b>(IgE)免疫球蛋白 IgE</b>					
Beckman Immage 800	IU/mL	29.9	12%	<5	<100
Beckman Coulter \ Immunonephelometry (INA)					
Siemens BN ProSpec	IU/mL	32.9	2.5%	<5	<100
Siemens \ Immunonephelometry (INA)					

<b>REF</b>	521-1	6x3mL	<b>EXP</b>	2025-01-01	<b>LOT</b>	31B2B49	更新日期:	2024-08
								2 / 3

		批号: 31B2B49			
项目\仪器\试剂方法	单位	均值 \ CV		Lab \ 点数	
<b>(IgG)免疫球蛋白 IgG</b>					
Siemens ADVIA 1800/2400 DiaSys \ Immunoturbidimetry (ITA)	mg/dL	691	3%	38	4264
Beckman AU series chongqiong zybio \ OTHER	mg/dL	694	2.6%	<5	<100
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	mg/dL	684	5.2%	11	558
Beckman AU series Beckman Coulter \ Immunonephelometry (INA)	mg/dL	675	9.9%	<5	<100
KHB Polaris i2400 KHB \ Chemiluminescence Immunoassay (CLIA )	mg/dL	720	9.9%	<5	<100
<b>(IgM)免疫球蛋白 IgM</b>					
Siemens ADVIA 1800/2400 DiaSys \ Immunoturbidimetry (ITA)	mg/dL	80.4	2.6%	38	4312
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	mg/dL	86.7	9%	<5	142
Beckman AU series Beckman Coulter \ Immunonephelometry (INA)	mg/dL	82	9.9%	<5	<100
KHB Polaris i2400 KHB \ Chemiluminescence Immunoassay (CLIA )	mg/dL	96	9.9%	<5	<100
<b>(Kappa Light Chain)κ轻链</b>					
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	g/L	5.85	12%	<5	<100
<b>(Lambda Light Chain)λ 轻链</b>					
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	g/L	2.97	12%	<5	<100
<b>(PAB)前白蛋白</b>					
Mindray BS series OTHER \ OTHER	g/L	0.182	3.9%	8	380
Beckman AU series KINGSBIO \ Immunoturbidimetry (ITA)	g/L	0.206	5.6%	8	372
Beckman AU series chongqiong zybio \ OTHER	g/L	0.195	0.9%	<5	<100
Roche Hitachi/Hitachi Systems CONDOR-TECO \ Immunoturbidimetry (ITA)	g/L	0.337	11.1%	10	520
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	g/L	0.184	2.6%	<5	<100
KHB Polaris i2400 KHB \ Chemiluminescence Immunoassay (CLIA )	g/L	0.206	9.9%	<5	<100
Beckman AU series Beckman Coulter \ Immunonephelometry (INA)	g/L	0.21	9.9%	<5	<100
Roche cobas 4000/6000/8000 DiaSys \ Immunonephelometry (INA)	g/L	0.161	11.9%	<5	<100
TOSHIBA TBA TOSHIBA \ Immunoturbidimetry (ITA)	g/L	0.172	0.5%	<5	<100
<b>(RBP)视黄醇结合蛋白</b>					
Beckman AU series DongOu \ OTHER	mg/dL	2.62	4.7%	7	162
Siemens ADVIA 1800/2400 BIOSINO \ Immunoturbidimetry (ITA)	mg/dL	3.07	2.4%	38	2151
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	mg/dL	2.97	1.8%	<5	<100
Roche cobas 4000/6000/8000 WanTai \ Immunonephelometry (INA)	mg/dL	2.76	12%	<5	<100
<b>(RF)类风湿因子</b>					
Mindray BS series MedicalSystem \ OTHER	IU/mL	42.2	2.6%	8	380
Siemens ADVIA 1800/2400 DiaSys \ Immunoturbidimetry (ITA)	IU/mL	38.6	6.4%	38	4258
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	IU/mL	34.7	12.9%	10	502
Beckman AU series Beckman Coulter \ Immunonephelometry (INA)	IU/mL	44.4	9.9%	<5	<100
TOSHIBA TBA TOSHIBA \ Immunoturbidimetry (ITA)	IU/mL	40	1%	<5	<100
KHB Polaris i2400 KHB \ Chemiluminescence Immunoassay (CLIA )	IU/mL	43.7	9.9%	<5	<100
Siemens BN ProSpec Siemens \ Immunonephelometry (INA)	IU/mL	38.4	2.4%	<5	<100
<b>(TRF)转铁蛋白</b>					
Beckman AU series Reebio \ OTHER	mg/dL	126	2.2%	<5	<100
Siemens ADVIA 1800/2400 DiaSys \ Immunoturbidimetry (ITA)	mg/dL	125	4%	19	1031
Roche cobas 4000/6000/8000 Roche \ Immunoturbidimetry (ITA)	mg/dL	127	12%	<5	<100
Beckman Immage 800 Beckman Coulter \ Immunonephelometry (INA)	mg/dL	116	3.2%	<5	<100
<b>(β2M)β-2-微球蛋白</b>					
Beckman AU series chongqiong zybio \ Immunoturbidimetry (ITA)	ng/mL	2211	6%	9	454
Mindray BS series MedicalSystem \ OTHER	ng/mL	2374	5.3%	8	380
Beckman AU series DongOu \ OTHER	ng/mL	2497	5.1%	7	326

<b>REF</b>	521-1	6x3mL	<b>EXP</b>	2025-01-01	<b>LOT</b>	31B2B49	更新日期：	2024-08 3 / 3
------------	-------	-------	------------	------------	------------	---------	-------	------------------

		批号：31B2B49			
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
Roche Hitachi/Hitachi Systems MedicalSystem \ Immunoturbidimetry (ITA)	ng/mL	2546	2.5%	6	280
Beckman AU series chongqiong zybio \ OTHER	ng/mL	2050	0.6%	<5	<100
KHB Polaris i2400 KHB \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2070	9.9%	<5	<100
Roche cobas 4000/6000/8000 JS (Shanghai) \ Immunonephelometry (INA)	ng/mL	2410	11.9%	<5	<100
TOSHIBA TBA TOSHIBA \ Immunoturbidimetry (ITA)	ng/mL	1572	1.6%	<5	<100

注：此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来，仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。